



Don't use old adhesive foams, white on both sides, with the new white electrode grids.

The new electrode grids must be used only with new white-lightblue foams. The lightblue side is the skin side. New grids don't have the silicon paper, necessary with the old and more sticky foams.

Without silicone paper the new white grids are more easy to be cleaned by increasing their life. Refer to the foams instruction for further details or contact us if you are not sure about the foams to use.

New white electrode grid code example: GR08MM1305

- 08MM is the inter-electrode distance with measurement unit
- 13 is the number of electrode rows
- 05 is the number of electrode columns

otbioelettronica.it